






## Instructions for Downloading Internal Data

Step	Instructions	Graphic
<p><b>Note:</b> If there is a data card currently installed in your AED, skip to step 4, otherwise continue</p>		
<b>1</b>	Eject the battery pack from the AED by pressing the orange eject button on the side of the unit.	
<b>2</b>	Insert a data card, label side up (Defibtech item number DDC-6 or DDC-12) into the card slot above the battery opening. The data card will click into place and should be flush with the end of the card slot when installed properly.	
<b>3</b>	Reinsert the battery pack. Press until it clicks into place.	
<b>4</b>	Turn the AED on.	
<b>5</b>	Press and hold the on/off button for 5-10 seconds - until the voice prompts inform you that the data is being transferred	
<b>6</b>	During the data transfer, the LEDs on the face of the AED will blink. Upon completion, the LEDs will stop blinking and the unit will automatically turn off.	
<b>7</b>	Eject the battery pack from the AED by pressing the orange eject button on the side of the unit	
<b>8</b>	Remove the data card by pressing it in until it clicks and then allowing it to be partially ejected. Remove the data card the rest of the way from the card slot by pulling on it gently.	
<b>9</b>	Replace Battery.	



Thank you for reading this data sheet.

For pricing or for further information, please contact us at our UK Office, using the details below.



**UK Office**

**Keison Products,**

**P.O. Box 2124, Chelmsford, Essex, CM1 3UP, England.**

**Tel: +44 (0)330 088 0560**

**Fax: +44 (0)1245 808399**

**Email: [sales@keison.co.uk](mailto:sales@keison.co.uk)**

Please note - Product designs and specifications are subject to change without notice. The user is responsible for determining the suitability of this product.